


5 Create pin tucks

Fold the mask in half and press. Open the fabric and fold edges to the middle pressmark. Press and open. Then, by each pressmark create 1/2" pin tucks facing upwards. Sew along the outside of the entire mask using a 3/8" inch seam allowance.

Fold in half

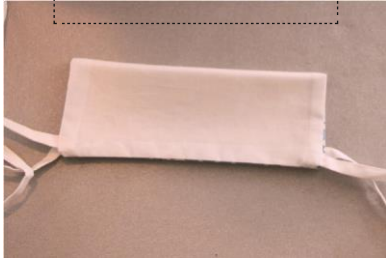

Fold edges to the middle pressmark




pin tucks



6 Fabric tie instructions

If you do not have elastic, you can make fabric ties. You basically make/use bias tape.

Cut four pieces of cotton fabric 1 1/2" wide and 16" long. Take one piece, fold in, and press one short edge 1/2". Then fold it in half lengthwise and press. Open and fold in both raw edges to the middle. Fold that in half and top stitch the tie. Repeat with other three pieces.

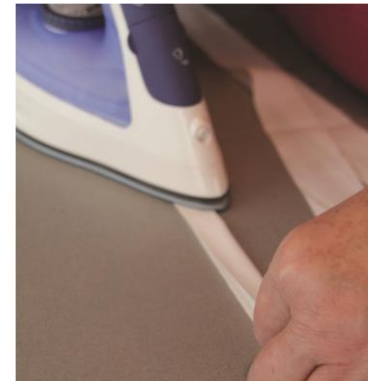

7

Zig-zag nose piece onto outside of mask, approximately 1/4 inch from top edge.



8 Packaging and shipping instructions

Please place **20 masks** in a Ziploc bag and mark the bag Surgical Masks. You can drop them off at any CAM Canada location or mail them to one of the following addresses. Please mark the package *Attention Receiving*.

Christian Aid Ministries Canada
1 Parkview Drive
Moorefield, ON N0G 2K0

Christian Aid Ministries Canada
3260 Bricker School Line,
Wallenstein, Ontario

